

ALMORE



bite registration sheets & tabs

Ideal for checking and maintaining accurate occlusal records. Bite Sheets & Tabs heat to “dead soft” 135°F (57.22°C) in one minute. An occlusion strip adds support to Bite Tabs while checking crown or bridge occlusion. Conforms in the mouth to every detail and sets by cool water or room temperature.

Almore Bite Registration Wax was developed to meet your needs of “Accurate Occlusal Records.” Our wax is so accurate that it records the details of the occlusion exactly. When trimming, avoid discrepancies by minimizing the wax bite so only the cusp tip indentations are present.

Heat in a thermostatically controlled waterbath at 135°F (57.22°C) for one minute. After obtaining jaw relation records, place the record in cold water to regain maximum hardness and to facilitate trimming of excess wax.

Bite Tabs: Almore Bite Registration Tabs are sized for quadrant bites. The tabs are ideal for bite relation records of single crowns, 3-4 unit bridges. The aluminum foil strip acts as a handle in which to grip the tab with as well as offering additional support and stability.

Heat in a thermostatically controlled waterbath at 135°F (57.22°C) for one minute. After obtaining jaw relation records, place the record in cold water to regain maximum hardness.

42600 SHEETS (1 LB. BOX - 18 SHEETS)

42602 TABS (125 PIECE BOX)