

Cleaning and Sterilization Instructions

Microfil Composite Instruments

Hager Worldwide Products are provided non-sterile

Ultrasonic Cleaner

- Place instrument in the Ultrasonic Cleaner until all bioburden is removed
- Use neutral PH solution for cleaning instrument
- Thoroughly rinse all solution from instrument
- Allow to dry

Instrument Washer

- Only Use Instrument Washer certified for use by FDA for Medical Dental Use
- Use neutral PH cleaner designed for use in Instrument Washer
- Run Washer for complete cycle

Sterilization

- Steam Sterilization is the recommended method
- Always consult and follow the sterilizer manufacturers instructions and recommendations

| Cycle Type | Minimum Temperature | Minimum Exposure Time | Drying Time |
|--------------|---------------------|-----------------------|-------------|
| Pre-vacuum | 121 C 240 F | 6 minutes | 30 minutes |
| Gravity Feed | 121 C 250 F | 30 minutes | 30 minutes |