

ALMORE



42900 CATALINA WATERBATH



42904 CATALINA²⁰⁰⁰ WATERBATH



42901
CATALINA INSERT TRAY



42905
CATALINA²⁰⁰⁰ INSERT TRAY

waterbaths

OPERATING INSTRUCTIONS

Model 42900: Used for softening wax, the Catalina Waterbath is a compact unit equipped with a stainless steel basin, detachable thermometer, and thermostat control. Adjustable temperature settings range from 100° F - 212° F (37.7°C - 100°C). Stainless steel basin measures 10 $\frac{3}{8}$ " x 6 $\frac{3}{8}$ " x 4".

Model 42904: Used for softening wax, the Catalina²⁰⁰⁰ Waterbath is a compact unit equipped with a stainless steel basin, detachable thermometer, and thermostat control. Adjustable temperature settings range from 100° F - 212° F (37.7°C - 100°C). Stainless steel basin measures 6 $\frac{3}{8}$ " x 6 $\frac{3}{8}$ " x 4".

FILLING THE UNIT

- **FILL UNIT WITH WATER BEFORE TURNING ON UNIT!**
- For best results, fill dish approximately 3½ inches of water. Take care not to overfill as water may overflow when using insert tray.

WATER

- For maximum purity distilled water may be used.
- When using high temperatures monitor water level closely as water may need to be added due to increased evaporation.



**DO NOT ALLOW WATERBATH TO RUN DRY
AS PERMANENT DAMAGE MAY RESULT!**



TEMPERATURE:

- The temperature will rise at a rate of 5°F (2.7°C) per minute.